**Instructions for use**  
**TSK PRIMECUT II BIOPSY NEEDLE**

**1 Intended purpose**

The TSK PRIMECUT II BIOPSY NEEDLE is a biopsy needle used to puncture the human body and collect tissue for examination, treatment, or diagnosis.

**2 Indications**

The device is indicated in patients requiring biopsy sample.

**3 Intended users**

The device is for use by medical doctors or health care professionals skilled in the procedure, which means it is not for use by anyone else. Only use by those medical doctors or health care professionals.

**4 Target patients**

The device is intended for use in patients 12 years of age and older of any gender, excluding cases of contraindications. The device should not be used in pregnant and lactating women.

**5 Performance-characteristics**

Insert into human body to collect a range of tissue samples including those from the breast, prostate, liver and kidney but excluding central nervous systems, for inspection, diagnosis, or treatment of patients.

**6 Clinical benefits**

The biopsy needle is a simple ancillary device, which is widely used in many clinical procedures; the clinical benefit of the device is to have a positive impact on patient management:

- Allow management of patients' medical conditions by providing a means of collecting tissue samples locally in order to meet patients' medical needs
- Allow medical professionals to manage medical conditions that require removal of tissue samples, and obtain tissue samples for diagnostic purposes.

**7 Undesirable side effects**

Possible adverse events associated with the use of the device are listed in the following:

- Pain/discomfort
- Pneumothorax
- Hypertension
- Bleeding
- Air embolism
- Hemodynamic instability
- Bruising
- Pleuritis
- Pleural effusion
- Death
- Pericardial effusion
- Diarrhea
- Edema
- Hematoma
- Urinary retention
- Hemorrhage
- Swelling/mass sensation
- Vasovagal reflex
- Pseudoaneurysm formation
- Hemothorax
- Subcutaneous emphysema
- Arteriovenous fistula due to puncture
- Short-term exacerbation of urinary symptoms
- Iatrogenic displacement of the tumor cells
- Lower urinary tract symptoms

**8 Contraindications**

- Do not use for patients who are receiving anticoagulant therapy, or have bleeding disorders or bleeding tendency.
- Do not use for the heart, the central circulatory system and the central nervous system

**9 Prohibitions**

- Do not reuse. [Otherwise, there is a risk of infection.]
- Do not resterilize. [Re-sterilization does not assure device sterility and could damage or degrade the device.]

**10 Warning**

**10.1 General precautions**

1.	Do not use the device under MRI. [Otherwise, there is a risk of MRI accidents.]
2.	Before using the device, read and follow the instructions for use.
3.	Do not use the device unless you are a medical doctor or a health care professional skilled in the procedure.
4.	If the patient or the device shows any abnormality, discontinue use immediately and take necessary measures.
5.	Check the device's appearance before use to ensure no damage (scratches, cracks, deformation, discoloration, and others).
6.	Do not use the device if the sterile bag damage (scratched, cracks, deformation, discoloration, and others) or the unintended opening before use are shown.
7.	Refer to the instructions if you use the device with pharmaceuticals or other medical devices.
8.	Patient management after biopsy depends on the biopsy technique and

	the physiological state of the individual patient. Monitor the patient's vital signs and strive to prevent and treat adverse events associated with the biopsy procedure.
9.	Do not use the device for any purpose other than those described in the "instructions for use."
10.	Do not modify the device.
11.	Operation practicing results in damage to the needlepoint and poor sharpness. Use samples for operation practice.

**10.2 Precautions for operation**


- Open and remove the device with the appropriate aseptic technique. [Otherwise, there is a risk of infection.]
- Use the device immediately after opening the package. [Otherwise, bacteria could multiply and cause infection.]
- Do not touch the needle directly with your hands. [Otherwise, there is a risk of infection.]
- Confirm the selection of gauge size and stroke length in advance. [Otherwise, there is a risk that you could not perform the appropriate procedure.]
- When using a biopsy needle guide such as an ultrasonic probe in combination, do not forcibly set the needle on a guide whose shape does not fit it. [Otherwise, there is a risk of device damage.]
- The amount of tissue sampling depends on the nature of the target tissue.
- Be careful when removing or attaching the protector. [Otherwise, there is a risk of needle stick injury.]
- The location of the puncture site should be confirmed by diagnostic equipment and palpation.
- Check the puncture route carefully when inserting the needle. [Otherwise, there is a risk of damaging unintended areas.]
- Do not deflect the needle during operation. [Otherwise, there is a risk of device damage.]
- Be careful not to let the needlepoint touch anywhere other than the intended contact surface. [Otherwise, there is a risk of the needlepoint damage.]
- There have been reports of air embolism due to puncture operations in the lungs, leading to cerebral infarction and myocardial ischemia. Thus, if the patient develops any of these symptoms, keep the patient's head in a low position, diagnose with CT and others, and take appropriate measures immediately. In severe cases, consider hyperbaric oxygen therapy without delay.
- Do not use the device to inject or drain fluids into or out of the body.
- After the procedure, take appropriate measures such as hemostasis.
- When wiping off the inner or outer needle, be careful not to get needle stick injuries and do not overload the device. Also, ensure that no foreign substance adheres to the needle parts. [Otherwise, there is a risk of device damage.]
- When performing repeated biopsies, check the appearance and ensure no damage (scratches, cracks, deformation, discoloration, and others) before use.
- Note that after biopsy, the blood and disinfectant (iodine tincture and others) could clot, and the levers could become stuck.

**10.3 Other precautions**

- The user must report all adverse events resulting from the use of the device to the manufacturer and to the competent authorities of the member state where the user or patient is.
- After use, dispose of the device safely and with care to prevent the risk of infection, following institutional, local, regional and national guidelines.

**11 Device specifications**

**11.1 Configuration (typical diagram)**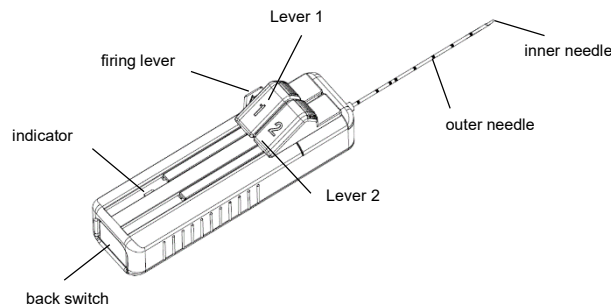

**11.2 Material**

Raw material for parts that contact the blood, body fluids, mucous membranes, and others is below.

- Inner and outer needles: stainless steel


**The device is ethylene oxide sterilized.**

**12 Instructions for use**


**12.1 Preparation for biopsy**

- 1) Open the sterile packaging of the device and aseptically remove the device.
- 2) Remove the protector from the needle of the device.

- 3) Pull the Lever 1 until it "clicks". When you have Lever 1 pulled, check the exposure of the sample notch of the inner needle and the marks on the outer one.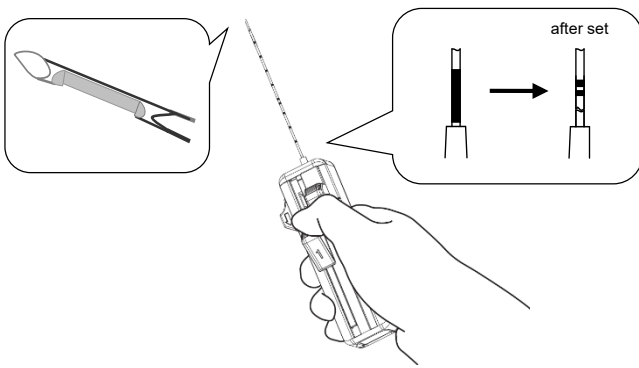


- 4) Then pull the Lever 2 until you hear a 'click' - check that the indicator is green after pulling the Lever 2.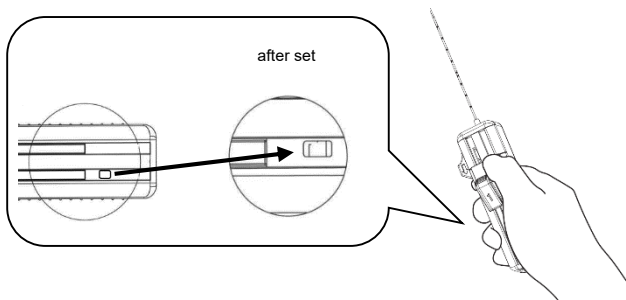

**12.2 Biopsy procedure**

Perform the biopsy procedure with an aseptic technique.

- 5) Disinfect the puncture site. Administer appropriate local anesthesia.
- 6) Puncture towards the marginal zone of the biopsy site. If there is resistance and the needle does not penetrate as expected during insertion, do not forcibly insert the needle and redo the puncture. [Otherwise, there is a risk of device damage.] Be careful not to accidentally activate the firing lever or the back switch during puncture.
- 7) When you pull the firing lever or press the back switch, the inner and outer needles move forward, and you can collect the tissue in the tissue sampling part. Operate the firing lever or the back switch securely. Do not operate them while pulling the Lever 1 or the Lever 2 with your fingers or others. [Otherwise, the device could not function properly.]

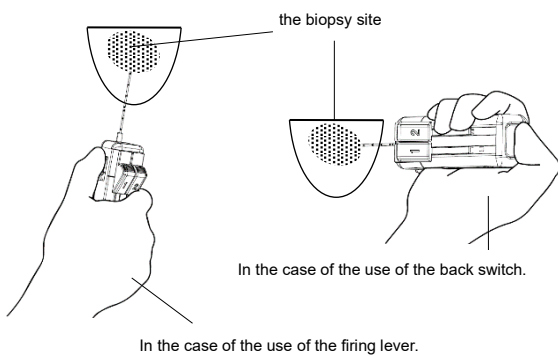- 8) Remove the needle from the puncture site. Do not operate levers during removal. [Otherwise, the device could not collect the tissue.]
- 9) After removing the needle, pull down the Lever 1 until it "clicks," and remove the tissue from the tissue sampling part. Be careful of the facing direction of the tissue sampling part. [Otherwise, you could drop off the tissue.]
- 10) If you repeat the biopsy after removing the tissue, pull down the Lever 2 until it "clicks."

**13 Storage**

- Keep dry. Keep away from direct sunlight, high temperature, and humidity. Store in a clean place.

**14 Expiry date**

- Refer to the expiry date printed on each package.
- Store properly and do not use the device after the expiry date.

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