

How to use



Be sure to set the safety switch to the LOCK position. Check that the sampling lever is in the closed position.

Grasp the end of the cover and pull the cover up from the main unit until you hear a "click". When you hear the sound, close the cover again.

Puncture and stop just before the biopsy site.

[ONE STEP METHOD]
[TWO STEP METHOD]

[ONE STEP METHOD]



POINT
The protrusion timing of the inner needle and the outer needle varies depending on the operating condition of the trigger. For reliable tissue collection, push the trigger firmly at once firing both needles almost simultaneously.

Set the safety switch to the "FIRE" position. Push the trigger in and the first firing sound will be the inner needle firing. Push the trigger button further to fire the outer needle. (A total of 2 firing sounds).

[TWO STEP METHOD]



Set the safety switch to the position between "FIRE" and "LOCK". Press the trigger to eject the inner needle only. (1 firing sound).

Set the safety switch to the "FIRE" position. Press the trigger again to eject the outer needle. (1 firing sound).



After removing the needle from the body, pull the sampling lever to the left and then down to reveal the tissue from the inner needle ready for collecting.



POINT
Be sure to return the sampling lever to the original position when performing a subsequent biopsy. (Lever up)

※Please refer to the latest package insert before use.


Product standard

Size		Product Code			
Gauge	Length	11mm	16mm	22mm	
11G	75mm	\	\	ACE-110752	
	14G	75mm	ACE-140751	ACE-140753	ACE-140752
14G	115mm	ACE-141151	ACE-141153	ACE-141152	
	150mm	ACE-141501	ACE-141503	ACE-141502	
	200mm	ACE-142001	ACE-142003	ACE-142002	
16G	75mm	ACE-160751	ACE-160753	ACE-160752	
	115mm	ACE-161151	ACE-161153	ACE-161152	
	150mm	ACE-161501	ACE-161503	ACE-161502	
16G	200mm	ACE-162001	ACE-162003	ACE-162002	
	18G	75mm	ACE-180751	ACE-180753	ACE-180752
		115mm	ACE-181151	ACE-181153	ACE-181152
150mm		ACE-181501	ACE-181503	ACE-181502	
18G	200mm	ACE-182001	ACE-182003	ACE-182002	
	20G	75mm	ACE-200751	ACE-200753	ACE-200752
		115mm	ACE-201151	ACE-201153	ACE-201152
150mm		ACE-201501	ACE-201503	ACE-201502	
200mm	ACE-202001	ACE-202003	ACE-202002		

Available in 3 types of penetration depths.



EOG Sterilization 5 pcs/box
Trade name: Automatic biopsy needle ACECUT
Single-use Class II Controlled medical device

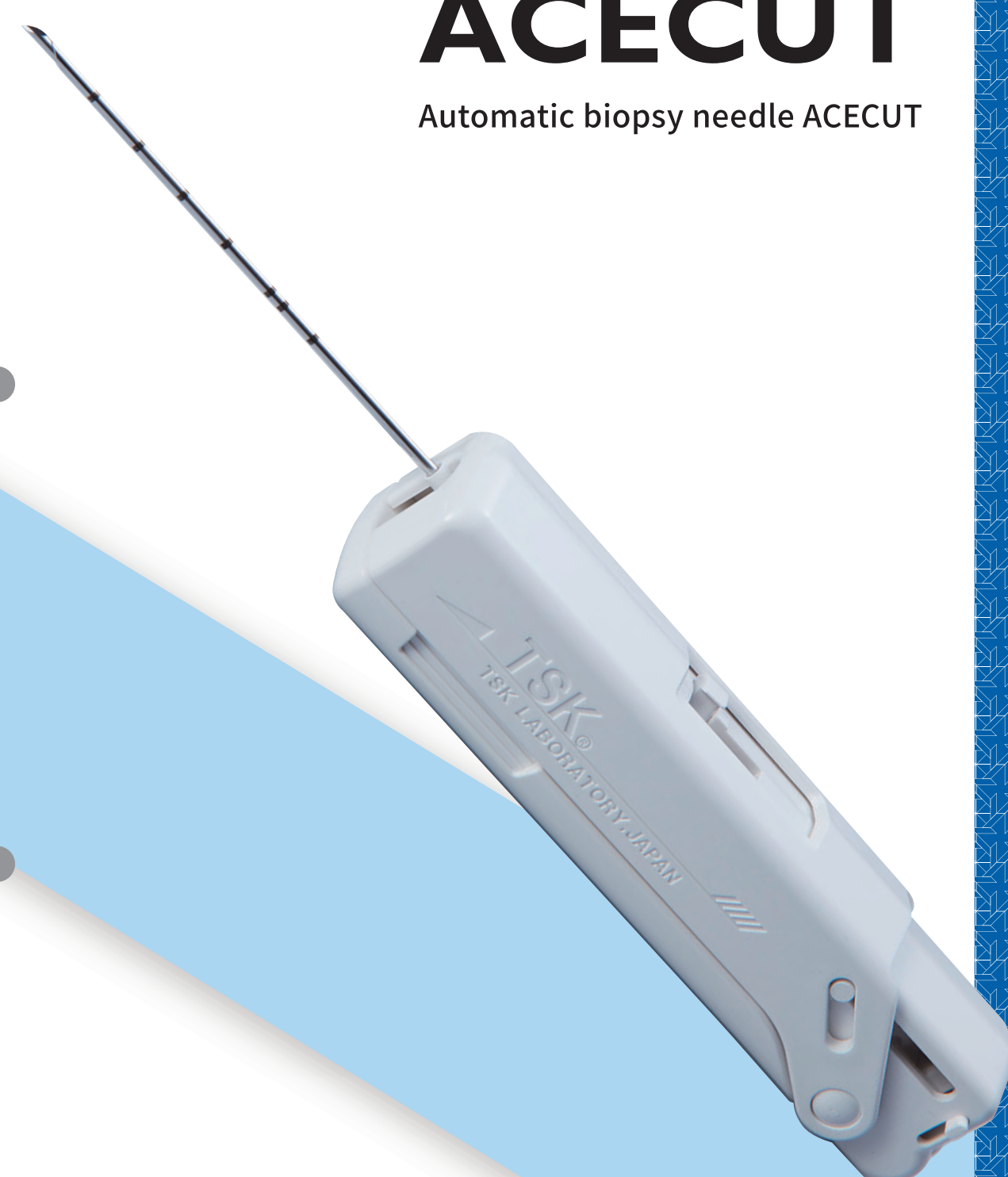

ACECUT


Automatic biopsy needle ACECUT



ACECUT

Automatic biopsy needle ACECUT

The ACECUT automatic biopsy needle is a single-use biopsy needle that is equipped with a function that can individually fire the inner and outer needles at a timing suitable for the practitioner's procedure.

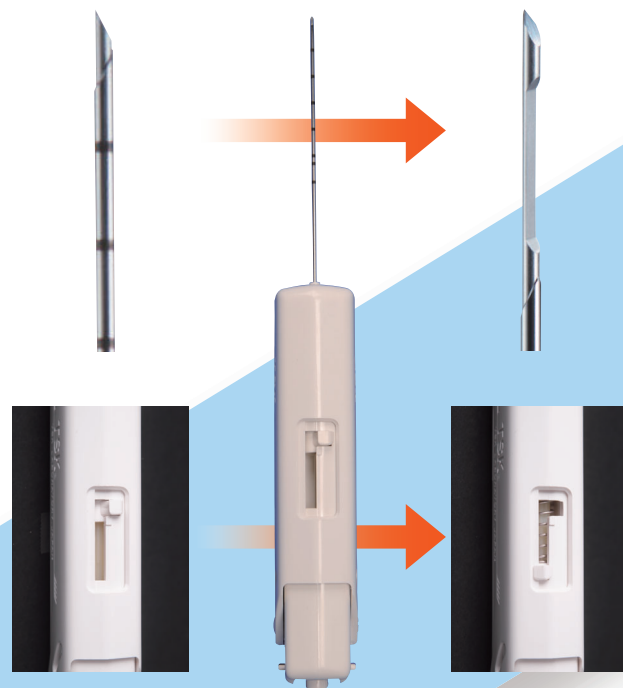

Sampling lever

Simply pull the sampling lever down to the open position to easily expose the tissue sampling area with one hand.

For automatic biopsy needle ACECUT TSK guide needle

Echo-compatible coaxial guide needle. ※Please inquire separately for size and price.



Depth marker

Allows visual confirmation of puncture depth.

Lines are placed at 1cm intervals, and double lines are placed at 5cm intervals.

TWO STEP method available

ADVANTAGE

After firing the internal needle under ultrasound, it can be confirmed that the needle is in the correct biopsy site.



Cover

Safety switch

A lock function is provided to prevent accidental firing.

Trigger